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Please read these instructions carefully before use!

Ticks mouse is a new product on the market for the fast, safe and contact-free removal of ticks (with storage possibility). This product leads the way as an innovative and multifunctional system for the removal of ticks. No other product currently on the market offers the possibility of collecting ticks in an integrated compartment specially designed for this purpose. Once the collection process is completed, all the parasites can be disposed of at once. This enables you to avoid the spread of the parasites in your living room or barn. At the same time, you also have a transport container in case you intend to take the ticks to a doctor for further examination (e.g. rapid screening for tick-borne diseases).

Ticks mouse effortlessly removes the parasites without applying pressure to the tick's gut. This prevents the reflux of tick saliva, which contains several irritants and sometimes microorganisms that may cause allergies or infections. Tweezers and similar instruments apply pressure to the tick. Ticks mouse removes the parasites without any contact with the skin, and leaves no mouth parts behind in the skin. It is also suitable for animals with long fur. We highly recommend that you use Ticks mouse if you own dogs, cats or horses, and if you are at risk of tick bites due to your profession (hunter, forestry worker, etc.).

SUITABLE FOR BOTH HUMANS AND ANIMALS: Ticks mouse effectively removes ticks from humans (children, adults) and animals (dogs, cats, horses, etc.). This makes it a must-have product for every household.

CLEAN: Parasites are collected in the integrated compartment – one by one – and removed or taken to a doctor for further examination.

CONTACT-FREE: At no point does your skin come into contact with the parasites, neither during their collection nor during their removal.

SAFE: It is virtually impossible to remove a tick incorrectly with the unique system of Ticks mouse.

HYGENIC: By boiling Ticks mouse in a water bath, cleaning is simple and sterile.

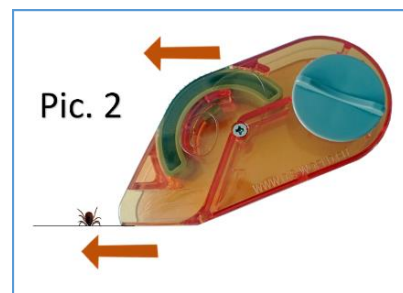
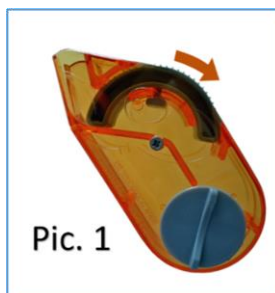
GERMAN DEVELOPMENT: Developed in Germany, Ticks mouse is patented.

Even at the slightest touch, ticks have the tendency to inject their secretion, which can be infected with various viruses, into their host. This makes it important to remove the tick and its head quickly and painlessly. The process of removing the tick should be successful at the first attempt. Once the parasite has locked itself into the skin, it is still generally very small. Removing a very small parasite from the host entails the risk of not catching it straightaway and thus not being able to remove it at all. Most accidents (head detachment, infections, etc.) occur while trying to remove the tick. That is why safety should always take priority over quick removal. If you are unsure about removing the tick quickly because of its small size, consult a qualified specialist (doctor or vet). You can leave the parasite in the body of the host until it has reached a size at which it is safe to remove. However, this also increases the risk of the host being infected with the secretion of the parasite.



Ticks mouse is very easy to handle but, just like any other instrument, must be used correctly. When used improperly, accidents (head detachment, infections, etc.) may occur. Should this be the case, consult a doctor or a vet. Once a tick has been removed, we highly recommend that you have it examined by your doctor for any viruses. This particularly applies if the parasite has been removed from a human.

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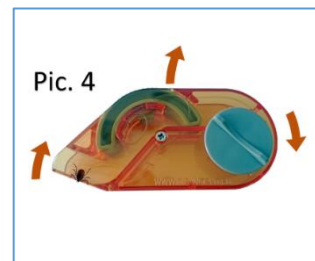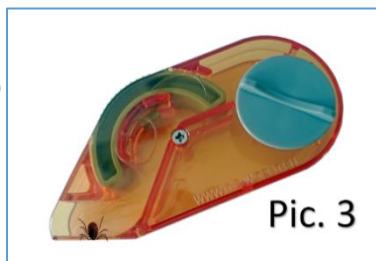- Pic. 1 Close the crescent slider (use your thumb to turn slider in the direction of the arrow)
- Pic. 2 Place the mouse on the skin, push the V-section centrally in the direction of the tick
- Pic. 3 Now the mouse is under the head of the tick
- Pic. 4 Simply lever out the tick (move the mouse in the direction of the arrow)
- Pic. 5 Position the mouse vertically, the tick falls into the bottom of the crescent slider
- Pic. 6 Open the crescent slider, the tick falls into the compartment
- Pic. 7 Close the crescent slider, the tick is then secured in the compartment
- Pic. 8 Once completed, open the cap lock (turn to the left) to dispose of all ticks at once

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